
Universal Alkaline Phosphatase Immunostaining Kit (For Mouse and Rabbit Primary Antibodies)

Catalog Number: KA50F
Document #: DS-6002-A
Effective Date: 05/01/2015

Intended Use
For In Vitro Diagnostic Use

****This data sheet is applicable to all sizes (volume) of product. Actual volume is indicated on vial.**

Summary and Explanation

This kit is designed to label specific primary antibodies immunohistochemically on tissue sections. The immunohistochemical protocol defined in this brochure is just a guideline. We encourage the individual laboratory to optimize its own protocol based upon tissue fixation conditions, primary antibodies employed, and the user's experience. These reagents were tested and quality controlled using tissue sections, however, they can also be optimized for cell smears and cytospin preparations. The linker reagent is a cocktail of biotinylated anti-mouse and anti-rabbit immunoglobulins, capable of labeling mouse and rabbit primary antibodies.

Principles of the Procedure

The high affinity of the non-covalent interaction between biotin and streptavidin (1×10^{15}) forms the basis for this immunostaining kit. It requires the formation of an irreversible and specific linkage between biological macromolecules. The immunohistochemical applications of the interaction between avidin and biotin were introduced by Bayer et al. (1979), who described techniques for generating active biotinyl compounds such as biotin-N-hydroxysuccinimide and biotin hydrazine and for conjugating them to various organic compounds, including immunoglobulins and alkaline phosphatase (ALP). Streptavidin (SA) is a tetrameric protein (mol. wt. $4 \times 15,000$), isolated from the actinobacterium *Streptomyces avidinii* (Chalet & Wolf, 1964). Streptavidin can bind to four molecules of biotin. Streptavidin gives superior results compared to avidin because its isoelectric point is closer to neutral pH, whereas avidin is positively charged at physiological pH. Streptavidin does not carry any carbohydrate side chain, whereas avidin is composed of 70% of carbohydrate. Because of this, SA does not have the tendency to bind non-specifically. Primary antibodies bind to the target antigens in the tissue sections and conjugated secondary antibodies bind specifically to these receptor antibodies. A biotin conjugated secondary antibody, in turn, is traced by a streptavidin conjugated enzyme and can be visualized by an appropriate substrate/chromogen.


Introduction

Immunohistochemical techniques are spreading rapidly and the practice of anatomic pathology has undergone a revolutionary change since the development of these procedures (Nadji & Morales, 1983). Because of their versatility, sensitivity, and specificity, immunoperoxidase stains are invariably the best stains when appropriate antibodies are available. With the ever increasing number of antibodies against cellular antigens, immunoperoxidase techniques now provide a powerful tool to resolve a wide array of diagnostic pathology. All immunohistochemical techniques require the specific antibody employed to be labeled that they can be easily seen when attached to cellular antigens. At the same time the sensitivity of the immunoperoxidase techniques are central to wide variety of specific antigen localization. Alkaline phosphatase based kits are of special value for staining tissues that have high endogenous peroxidase activity. Alkaline phosphatase is unaffected by endogenous peroxidase and therefore results in cleaner background. Our kit is based on direct SA-ALP conjugate technology.

Reagents Supplied

- | | |
|----------|--|
| Bottle 1 | Alkaline Phosphatase Enhancer: 10mL clear solution.
Used to increase the intensity of signal. |
| Bottle 2 | Linker Reagent: 10mL clear yellow solution of biotinylated anti-mouse and anti-rabbit immunoglobulins. |
| Bottle 3 | ALP Tracer Reagent: 10mL clear cherry red solution of conjugated streptavidin alkaline phosphatase. |

Diagnostic BioSystems
6616 Owens Drive
Pleasanton, CA, 94588
Tel: (925) 484 3350
www.dbiosys.com

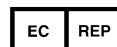

Emergo Europe
Prinsessegracht 20
2514 AP, The Hague
The Netherlands
Tel: (31) (0) 70 345 8570



-
- Step I Removal of paraffin wax: Deparaffinize tissue sections according to the established procedure in your lab and bring tissues to wash buffer.
- Step II Endogenous Alkaline Phosphatase Blocking: This is an optional step and should be performed only if tissue is suspected to have high endogenous alkaline phosphatase activity. Apply enough drops of Alkaline Phosphatase Blocker to cover tissue. Incubate for 5-10 minutes at room temperature.
- Step III Washing: Drain off excess reagent. Rinse with wash buffer three times for 1 minute each time. Drain off excess buffer and carefully wipe slide around the tissue to remove excess buffer from the glass leaving the tissue wet.

Immunostaining Protocol I

This protocol is recommended for optimally fixed tissues with abundant antigens in the tissue and for high affinity primary antibodies.

- Step I Primary antibody: Apply enough drops of primary antibody to cover the tissue section. Incubate according to the manufacturer's recommended conditions. Wash and wipe slides as described above.
- Step II Linker reagent: Apply enough drops of linker reagent to cover the tissue section. Incubate for 10 minutes at room temperature. Wash slides as described previously.
- Step III Tracer Reagent: Apply enough drops of tracer reagent to cover the tissue section. Incubate for 10 minutes at room temperature. Wash slides as described previously.
- Step IV Substrate/Chromogen: Apply working chromogen solution for 5-10 minutes at room temperature for color development. For best results, look under the microscope for signal development. Once desired signal to noise ratio is achieved, stop the reaction by washing slides in wash buffer. Note development time and follow it during subsequent incubations.

Immunostaining Protocol II

This protocol is recommended for less than optimally fixed tissues with low antigenic density in the tissue and for low affinity primary antibodies.

- Step I Primary antibody: Apply enough drops of primary antibody to cover the tissue section. Incubate according to the manufacturer's recommended conditions. Wash slides as described previously.
- Step II Linker reagent: Apply enough drops of linker reagent to cover the tissue section. Incubate for 20 minutes at room temperature. Wash slides as described previously.
- Step III Tracer Reagent: Apply enough drops of tracer reagent to cover the tissue section. Incubate for 20 minutes at room temperature. Wash slides as described previously.
- Step IV Substrate/Chromogen: Apply working chromogen solution for 5-10 minutes at room temperature for color development. For the best results, look under the microscope for the signal development. Once desired signal to noise ratio is achieved, stop the reaction by washing the slide in wash buffer. Note development time and follow it during subsequent incubations.
- Step V Wash slides and counter stain with an appropriate counter stain. Mount and observe staining under the microscope.

Enhanced Staining

If additional signal is desired, treat tissue sections with Alkaline Phosphatase Enhancer for 1 minute after Step III. Drain off excess enhancer and add substrate/chromogen solution without any wash.

References

- (i) Bayer et al., Method Enzymol. 62, 308 (1979).
- (ii) Chaiet & Wolf, Arch. Biochem. Biophys. 106, 1 (1964).
- (iii) Giorno, Diag. Immuno. 2, 161 (1984).



-
- (iv) Hsu et al., J Histo. Cyto. 29, 577 (1981).
 - (v) Nadji & Morales, Anat. pathol. 14, 767 (1983).
 - (vi) Nagle et al. J Histo. Chem. 31, 1010 (1983).
 - (vii) Larsson, Applied Immunohistochem. 1, 2, (1993).
 - (viii) Petrusz & Ordronneau, In Polak & Van noorden eds., Immunocytochemistry: Practical applications in pathology and biology. Bristol wright-PSG, 212, (1983).

